


Download app & Signup

1
Download
Dr-Neodocs app

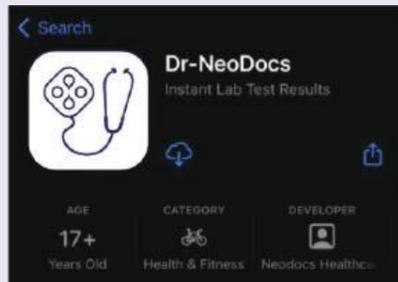


For doctors & clinic staff




Download on the
App Store

GET IT ON
Google Play


2
Enter phone
number & OTP

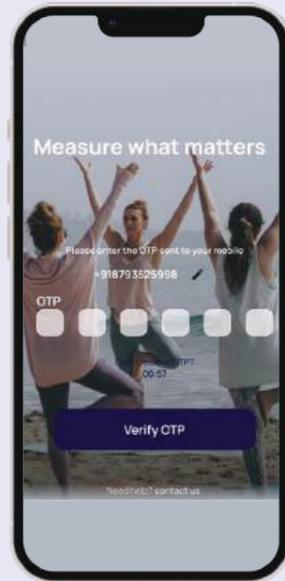

3
Enter first &
last name

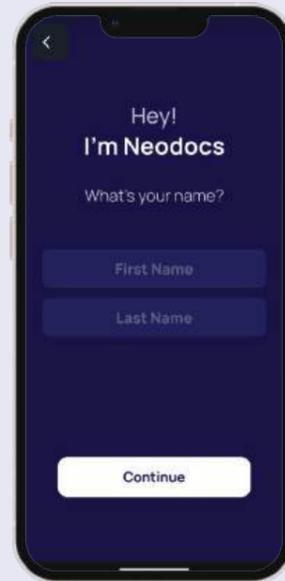


4
Enter organization
password

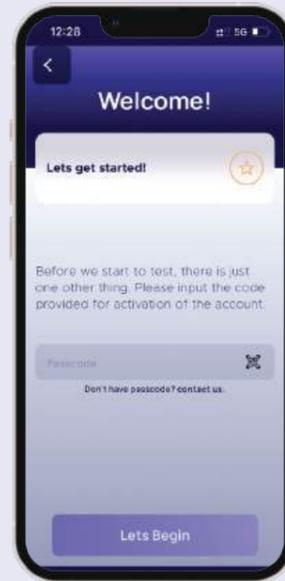


Note: If you don't have the password or face any difficulty, Please contact us at: +91 9987339111

How to take the test

1
Click on
"Add New Test"

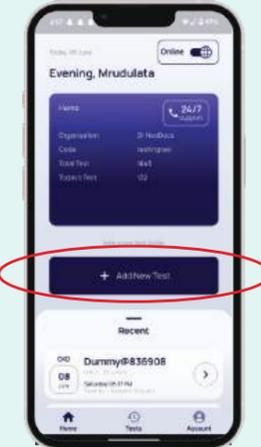

2
Enter patient details
Name, Age, Gender
Phone number & Case id (Optional)

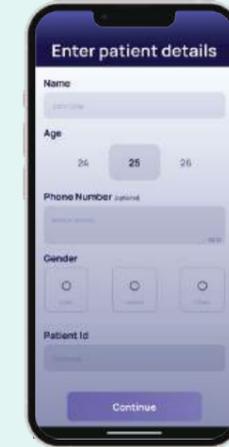

3
Request the patient to
collect a urine sample



Scan this QR to
watch the video
on "How to take
uACR test"

4
Take out the
testcard from pouch



5
Dip the bottom part of testcard
in urine completely for 1-2 sec



6
Place testcard on control pad
and start timer immediately



7
Take a clear
picture of the testcard




8
Results in
30 seconds

Investigation	Observed Value	Units	Biological Reference Interval
ACR	42 (POS)	mg/dl	Normal < 30 mg/dl Abnormal ≥ 30-100 mg/dl Severe ≥ 100 mg/dl
Microalbumin	210.0	mg/dl	< 30 mg/dl
Protein Albumin	300 (H)	mg/dl	< 30 mg/dl
Glucose	Negative	mg/dl	Negative
Cholesterol	52	mg/dl	50-200 mg/dl
Bilirubin	0.0	mg/dl	< 1.2 mg/dl
Urea	Negative	mg/dl	Negative
Uric Acid	Negative	mg/dl	Negative
Leucocytes	0-10	per field	0-10
RBC	0	per field	0
WBC	0	per field	Negative

Do's & Don'ts

Make sure you download
the right app - Dr-Neodocs

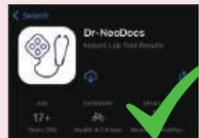

Make sure you select the
correct organization



Dip the bottom part of
testcard completely




Click photo immediately
(Once the timer ends)



Don't open the pouch before
collecting the urine sample



Don't re-use the
testcard

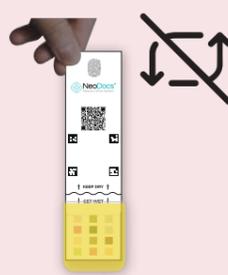

Always perform test on
flat surface



Make sure the room is
brightly lit



Note: All these steps are available in app as well